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■ *Its products being distributed to over 120 countries, the Japanese manufacturer MANI is one of the largest companies in the Asian medical industry, and its extensive portfolio includes endodontic instruments. Founded in 1956, the company proudly looks back on a long history of research, innovation and quality. In this interview, Shuhei Takahashi, manager of MANI's European business development at its subsidiary the Gesellschaft für dentale Forschung und Innovationen (society for dental research and innovation), and Dr Ralf Schlichting, board member of the Deutsche Gesellschaft für Endodontologie und zahnärztliche Traumatologie (German society for endodontics and dental traumatology), talk to Dental Tribune International about MANI's dental products and its*

◀ Die JIZAI-Feilen von MANI. ▶ The JIZAI files from MANI.

*JIZAI endodontic file system in particular.*

**Mr Takahashi, MANI has its own dental division with a focus on endodontics. Could you briefly introduce this product portfolio?**

*We have been developing and producing medical stainless-steel needles for many years. After gaining significant experience with our unique technology, we decided to apply our expertise to high-quality products for the dental market. Today, MANI offers endodontic instruments and rotary burs. Our endodontic files, such as MANI K-Files, H-Files, RT files and D Finders, are established and recognised MANI brand names around the world.*

**JIZAI is MANI's latest rotary nickel-titanium (NiTi) system for root canal preparation. How was this system developed?**

*The JIZAI file system was developed by our research and development department in close collaboration with endodontists from universities and specialist practices. Naturally, clinical research played a central role in the development with regard to file design and the unique heat treatment of the JIZAI files, and we worked closely with experienced endodontists in order to enhance the clinical performance and feel of the files. In particular, MANI is proud of the fact that the first university studies were carried out even before the official launch of the JIZAI files, the results of which were very positive overall compared with our competitors. We wanted to ensure that we could offer a well-engineered product that had already been scientifically tested.*

**Dr Schlichting, the dental market offers a variety of file systems for root canal preparation. What are the features of an optimal file system in your opinion?**

*Endodontic treatment is very challenging, as we have to deal with many different and sometimes very demanding anatomies. The second challenge we face is achieving the biological goal of endodontic therapy, namely the removal of infected tissue and complex microbiological structures such as biofilm. The preservation of natural teeth largely depends on how well we are able to achieve this biological goal. I doubt that there*

*will ever be the optimal file system. However, there are some requirements that I would like to have fulfilled: a file system must ensure maximum treatment safety, be as flexible and minimally invasive as possible, allow retention of the original root canal anatomy and be easy to integrate into the treatment workflow.*

**What is your experience with the JIZAI files in practice?**

*I have been familiar with the JIZAI file system for a very long time. Owing to only positive experiences, we now treat almost all cases with this file system in our practice, which is limited to endodontics. There are three aspects that particularly impress me: the high degree of treatment safety, the outstanding flexibility—especially with larger size files—and the very precise retention of the canal anatomy, which allows us to preserve as much tooth substance as possible. These factors have also been demonstrated in independent studies.*

**What distinguishes the JIZAI files from other file systems that you use in daily practice?**

*In my opinion, one of the major differences is the excellent flexibility, even with larger files such as the 30/0.04 and 35/0.04. Small size files of up to about ISO 25 from other manufacturers have sufficient flexibility; however, flexibility decreases dramatically with larger size files. Therefore, it is recommended to prepare curved canals only with small ISO size files. With regard to the biological goal of endodontic treatment, this approach is wrong in my opinion. Numerous studies have shown that a certain minimum preparation size is necessary to achieve adequate cleaning, both mechanically and with disinfectant solutions. With the JIZAI system, even very curved canals can be shaped up to 30/0.04 and in most cases even up to 35/0.04. Another advantage is that all the files can be used at the same speed of 500rpm and a torque of 3Ncm, eliminating the need to switch between different files. ◀*

Editorial note: More information about MANI can be found at [www.mani.co.jp/en](http://www.mani.co.jp/en). IDS visitors can learn more about MANI's portfolio and the new JIZAI files at the company's booth (#H019) in Hall 10.1.

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◀ Dentist Dr Ralf Schlichting (left) is a convinced user of MANI's JIZAI files. Together with Shuhei Takahashi, manager of MANI's European business development at its subsidiary the Gesellschaft für dentale Forschung und Innovationen (society for dental research and innovation), he introduces the system in this interview.

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