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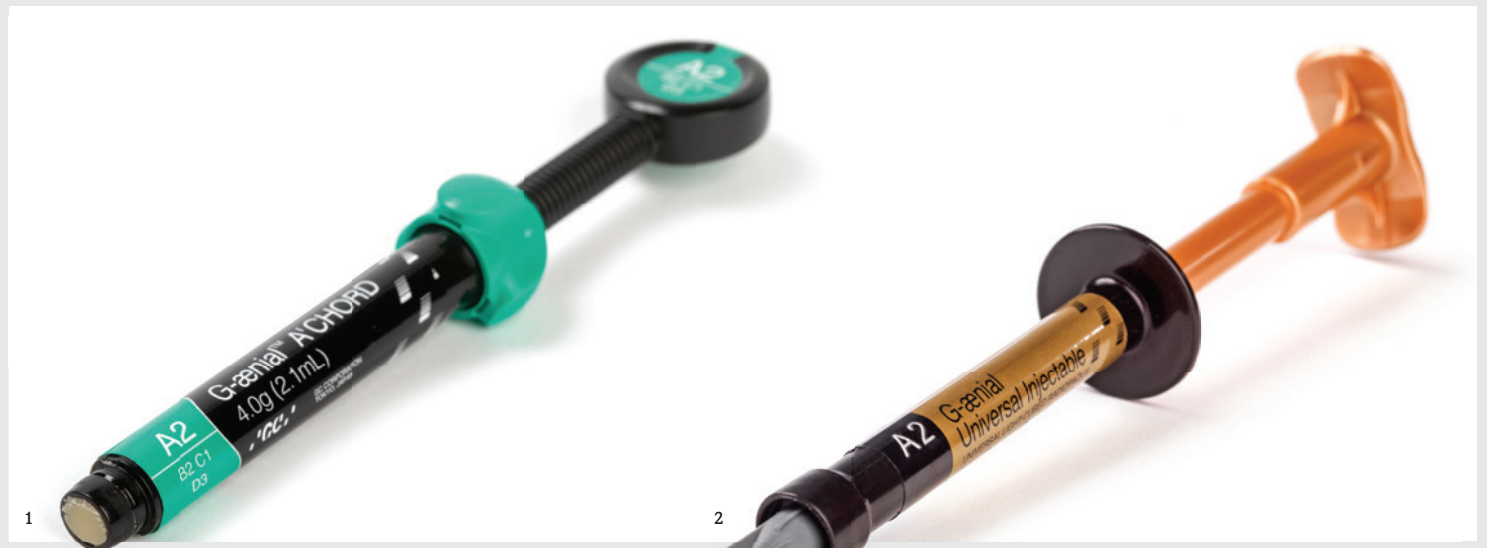
■ At GC, it is understood that even the smallest changes can make a significant difference for patients. This is why the “Small changes, big smiles” initiative focuses on the innovative Smile Sculpting technique. This approach empowers dental professionals to achieve favourable aesthetic results using a single shade and a freehand technique, all within one visit.

It is an advanced method for transforming smiles efficiently, predictably and with minimal intervention.

Smile Sculpting is a straightforward but highly effective technique that enables dentists to close gaps, eliminate black triangles, and seamlessly restore worn teeth. By using a single universal shade, practitioners can work more efficiently without compromising the natural appearance of the restoration. The freehand approach provides complete control over shaping, offering an intuitive and efficient solution for a variety of cases. Combined with GC's advanced materials, this technique ensures high-quality results while preserving the natural tooth structure.

At the core of this approach is G-ænia A'CHORD (Fig. 1), GC's universal restorative material, which combines simplified shade matching, excellent handling, and a seamless,

natural finish. Its superior blending properties allow dentists to achieve aesthetic restorations quickly and efficiently. Complementing this is G-ænia Universal Injectable (Fig. 2), which offers exceptional strength, wear resistance, and precision for freehand sculpting. Combined, these materials support efficient, minimally invasive procedures that meet high aesthetic standards. For optimal bonding during the Smile Sculpting procedure, G-Premio BOND (Fig. 3) is applied to ensure a strong and durable adhesion between the tooth and the restorative material. Post-procedure, the application of GC MI Paste Plus (Fig. 4) aids in remineralising enamel and reducing sensitivity, enhancing the overall success of the Smile Sculpting technique.

GC offers a dedicated Smile Sculpting course, designed to help dental professionals master this technique and achieve optimal outcomes. The practical training covers streamlined workflows, reduced chair time, and maintaining top standards of quality and aesthetics. Participants will gain hands-on experience and learn how to incorporate this innovative technique into their practice, enhancing patient outcomes with ease and efficiency.

To experience Smile Sculpting firsthand, dental professionals are encouraged to join GC's special hands-on courses and live demonstrations. Led by internationally recognised dentists and technicians at the speakers' corner, these events showcase how GC's solutions and techniques enable small changes that create big smiles—and even greater patient satisfaction. ◀◀

GC Europe, Belgium


www.gceurope.com

**Hall 11.2,
Booth N010/O039**

AD

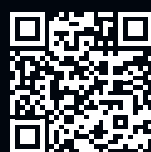Sterify Gel
For the Treatment
of Periodontal Disease

Discover at IDS!
Hall 10.1
Stand C-070

HuFriedyGroup
The Best In Practice

©2025 Hu-Friedy Mfg. Co., LLC. All rights reserved. 0225

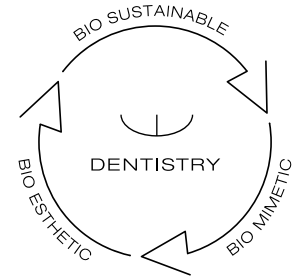
edelweiss SYSTEMS

- VENEER
- OCCLUSIONVD
- COMPOSITE
- POST & CORE
- PEDIATRIC CROWN
- PEDIATRIC VENEER
- CAD/CAM BLOCK

shaping the future of dentistry

PEDIATRIC CROWN

REVOLUTIONISE PEDIATRIC DENTAL CARE

ADVANTAGES

- Unrivalled innovation
- Natural abrasion
- Superior aesthetics
- Bio-mimetic
- Minimally invasive
- Optimal fit and function
- Bisphenol A free

BEFORE

AFTER

PEDIATRIC VENEER

THE FUTURE OF PEDIATRIC DENTAL CARE

WORLD NOVELTY

ADVANTAGES

- Minimally invasive, easy and efficient application
- World's first veneers for children setting a new standard in pediatric dental care
- Blend of innovation, aesthetics and function that will revolutionise how we care for our children's smiles

BEFORE

AFTER

Convince yourself and try our products at a live hands-on station at the edelweiss booth. We are happy to welcome you at Hall 11.3, Stand B040 – C041.

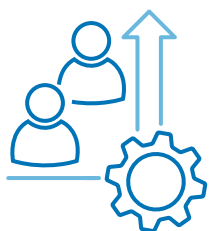
www.edelweissdentistry.com

Because so many **rely on you,**
you can **rely on us**


Practices are unique and constantly evolving.
Rely on Us for customized solutions
that empower optimal patient and practice outcomes.


Business Solutions

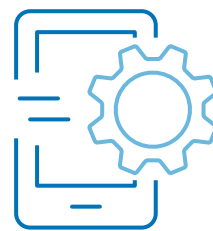Clinical Solutions

Technology Solutions

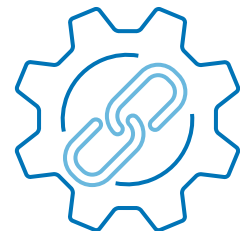Supply Chain Solutions

Looking for solutions?
visit: www.henryschein.com

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