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■ Shade determination is carried out with the utmost care. However, it can still happen that the final restoration appears lifeless when seated, with too little red and insufficient chroma. This is often due to an optical illusion. It occurs because shade tabs are typically assessed without the pink gingival tissues, against the dark background of the oral cavity. As a result, incorrect shade choices may be made, which cannot be corrected during laboratory fabrication.

The SHOFU GUMY-V gingiva-coloured shade tab holder allows for a more accurate simulation of the gingival effect. It enhances objectivity in shade determination, improves communication between dentists and dental technicians, and reduces mis-

understandings as well as time-consuming adjustments and remakes.

The SHOFU GUMY-V facilitates precise intra-oral shade determination using the VITA Classical A1–D4 shade guide and helps to avoid a greyish effect or insufficient chroma in the final restoration. It is suitable for both dental clinicians who wish to determine tooth shades chairside and ensure accuracy, as well as for dental technicians who take on shade matching as a service, for example, in the fabrication of anterior crowns.

Application is straightforward. First, the appropriate GUMY-V is selected. The tooth shade is then determined without the gingiva, followed by a second check using the gingiva-coloured GUMY-V. If necessary, the shade can be corrected before proceeding.

Digital photo documentation with the EyeSpecial C-V

In addition to visual shade matching with a shade guide and SHOFU GUMY-V, thorough photo documentation is recommended. A photographic comparison of the natural tooth shade with corresponding shade tabs provides additional information. Furthermore, the dental technician gains valuable insights into tooth shape and unique characteristics such as enamel cracks or white spots.

The EyeSpecial C-V (Fig. 1) dental camera features a dedicated Isolate Shade mode for image documentation with and without gingiva. In this mode, up to three images can be captured: one in true colour, one with automatic masking of the gingival areas, and one in black and white for brightness assessment. After selecting the appropriate mode, the camera takes care of the rest – providing a significant advantage for collaboration with the dental laboratory.

At the SHOFU booth, visitors will have the opportunity to experience the latest products in live demonstrations. ◀◀

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