OMNICHROMA: 1.000 FARBEN WEISS IN 1 SPRITZE
OMNICHROMA: 1,000 SHADES OF WHITE IN ONE SYRINGE

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■ *Dentist Dr Kevin M. Brown from Bellevue in Washington in the US knows his patients expect the very best, from quality care to outstanding results. But, as he knows very well, investing in the highest quality products and materials can be costly. When Brown first learnt about OMNICHROMA, he was fascinated by its use of structural colour as the main colour mechanism, enabling the composite to adapt to any tooth shade. OMNICHROMA can transform to an A1, a D3 or any other shade the tooth happens to be. “Having evaluated the OMNICHROMA composite for some time now, I can see that versatility is just one of the many benefits it*

offers,” he explained. “From a practicality standpoint, OMNICHROMA is a game changer. One composite that can work for over 90 per cent of my patients is a dream come true. After OMNICHROMA was officially launched in February 2019, we are now able to limit our order to one composite and our overhead will be decreased significantly,” he pointed out. Most composites today depend on the chemical colour of the dyes and pigments added to the resin material to emulate certain shades of human teeth. These composites are limited in their ability to shade match. OMNICHROMA’s Smart Chromatic Technology is the first use of structural colour in composite dentistry as the main colour mechanism. As ambient light passes through the spherical fillers in OMNICHROMA, they generate red to yellow structural colour, the natural colours found in

all human teeth. The result is an unprecedented shade matching ability combined with excellent mechanical properties and high polishability. For the dental practice, that means simplified inventory management, reduction of composite shades (particularly those that only see incidental use), reduction of unused composite, and never being short-stocked on a shade. OMNICHROMA is recommended for direct anterior and posterior restorations, direct bonded composite veneers, diastema closure and the repair of porcelain/composite. ◀◀

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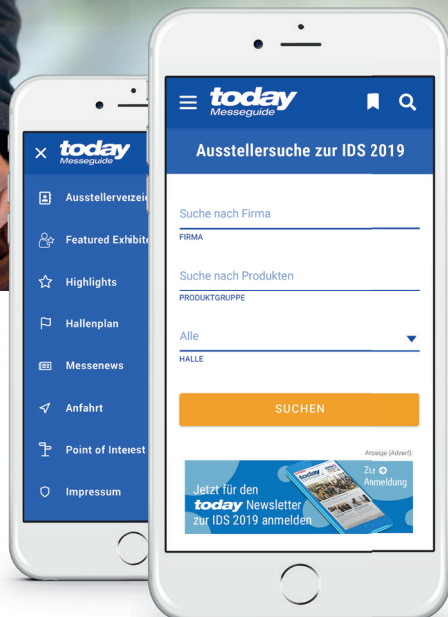
■ *Nowadays, an increasing number of restorations are being produced with monolithic or only partially veneered CAD/CAM-supported zirconia. However, zirconia does not generally possess fluorescence, which plays a role in the natural dynamics of colour and light. The VITA AKZENT Plus FLUOGLAZE LT spray now makes it possible to achieve a natural fluorescence and glaze with only one touch of the nozzle. The fine-spray application achieves a uniform thickness and homogeneous fluorescence across the entire surface. In just one firing, VITA AKZENT FLUOGLAZE LT helps to accelerate the digital workflow and supports the natural appearance of fully anatomical crown and bridge parts made of zirconia. ◀◀*

VITA Zahnfabrik, Germany
www.vita-zahnfabrik.com
Hall 10.1
Booth D010-E019

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© Maridav / Shutterstock.com